



sutherlandmedical

Introducing **Tubular-Form**TM

The ideal tubular
compression bandage:



COMPRESSION BANDAGES

Tubular-Form™

Tubular Compression Bandage

Tubular-Form™ is ideally designed for the treatment of orthopaedic and vascular conditions..

Tubular-Form™ is used for:

- Treatment of Venous insufficiency requiring low – medium compression
- Control of Oedema/Swelling of the limbs
- Soft tissue, joint injuries or rib injuries
- Application following the removal of casts

Features:

- **100% Australian Made**
- Clinically shown to heal Venous Leg Ulcers with 3 Layer compression therapy (references available)
- Product treated with Unique Low Fray Formula
- Easy to use Measuring Guides available
- Easy to use Applicators available

Measuring Guide for Application

Code	Size	Application	Roll	Circumference Measurement**
SM099	AA	Infant Limbs	10m	< 6cm*
SM100/SM200 ^	A	Infant Limbs	10m	7 -11cm*
SM101/SM201 ^	B	Small Limbs	10m	12 -14cm*
SM102/SM202 ^	C	Adult Limbs	10m	15 – 25cm*
SM103/SM203 ^	D	Large Limbs	10m	26 – 35cm*
SM104/SM204 ^	E	Adult Legs	10m	36 – 44cm*
SM105/SM205 ^	F	Large Legs	10m	45 -50cm*
SM106/SM206 ^	G	Large Thighs	10m	51 – 60cm*
SM107	H	Extra Large Thighs	10m	61 – 70cm*
SM108	J	Small Trunk	10m	71 -75cm*
SM109	K	Medium Trunk	10m	76 -80cm*
SM112	K2	Medium Large Trunk	10m	81 -84cm*
SM110	L	Large Trunk	10m	85 – 90cm*
SM111	M	Extra Large Trunk	10m	> 90cm*



*When applying to limbs, measure to fit the narrowest part. When applying to other areas of the body measure the affected area

^ 100 Series indicates Natural Colour 200 Series indicates Flesh Colour ****Guide is for Medium Compression; for Low Compression select next size bandage up.**

Fibre Content: Cotton, Elastodiene polyurethane and Polyester

Sutherland Medical Pty Limited

8 Kingston Town Close, Oakleigh, Victoria

Ph: 1300 664 027 Fax: 1300 664 028

Email: orders@sutherlandmedical.com.au

Web Site: www.sutherlandmedical.com.au