



sutherlandmedical

Introducing **Tubular-Plus**TM

A deluxe alternative for
skin protection:



PROTECTION BANDAGES

Tubular-Plus™

Tubular Padded Protection Bandage

Tubular-Plus™ is ideally designed to provide a protective layer in direct contact with the skin.

Tubular-Plus™ is used for:

- Protection of the skin prior to the application of fibre-glass & plaster of paris casts.
- Padding to Limbs under compression bandaging.

Features:

- **Australian made**
- Cotton 98% (Elastodiene / Nylon 2%)

Application:

It is important when measuring Tubular-Plus prior to application, that an amount of 2.5cm is allowed at either end of the bandage.

The 2.5cm of additional bandage will gently roll after being cut to prevent airborne particles from entering the wound.

After using **Tubular-Plus** for an extended period, the bandage will be subtly shaped to the limb it has been applied. Gently pull the bandage lengthways to remove the memory and the bandage will return to its original specifications.

| <u>Code</u> | <u>Size</u> | <u>Application</u> | <u>Roll</u> |
|-------------|-------------|--------------------|-------------|
| SM6629 | 5cm* | Small Limb | 5m |
| SM6630 | 7.5cm* | Medium Limb | 5m |
| SM6631 | 10cm* | Large Limb | 5m |
| SM6632 | 12.5cm* | X Large Limb | 5m |
| SM6633 | 15cm* | Large Thigh | 5m |



*Size = Unstretched Width

Fibre content: Cotton, Elastodiene, Nylon

Sutherland Medical Pty Limited

8 Kingston Town Close, Oakleigh, Victoria

Ph: 1300 664 027 Fax: 1300 664 028

Email: orders@sutherlandmedical.com.au

Web Site: www.sutherlandmedical.com.au