



sutherlandmedical

Introducing **Tubular-Rib**<sup>TM</sup>

A natural alternative for  
skin protection:



PROTECTION BANDAGES

# Tubular-Rib™

## Tubular Retention Bandage

**Tubular-Rib™** is ideally designed to provide a protective layer in direct contact with the skin.

### **Tubular-Rib™** is used for:

- Protection of the skin prior to the application of fiber-glass or plaster of paris casts.
- Protection of the skin under all types of compression bandaging.

### Features:

- **Australian made**
- 100% Cotton

### Application:

It is important when measuring **Tubular-Rib™** prior to application that an amount of 5cm is allowed at either end of the bandage. The bandage can be gently rolled onto the limb or torso allowing a split at the thumb site for arm casts. **Tubular-Rib™** can be applied independently using multiple layers or in conjunction with flat undercast padding.

After applying the first layer of the cast, and while it is still wet, roll back and smooth the **Tubular-Rib™** over the end of the cast to create a comfortable cotton cuff. Continue with the casting ensuring subsequent layers do not extend past the cuff.

<u>Code</u>	<u>Size</u>	<u>Application</u>
SM500	5cm*	Small Limbs
SM501	7.5cm*	Medium Limbs
SM502	10cm*	Large Limbs
SM503	15cm*	Extra Large Limbs
SM504	20cm*	Trunks



\* Unstretched Width (All rolls 20m)

Fibre content: Cotton

### Sutherland Medical Pty Limited

8 Kingston Town Close, Oakleigh, Victoria

Ph: 1300 664 027 Fax: 1300 664 028

Email: [orders@sutherlandmedical.com.au](mailto:orders@sutherlandmedical.com.au)

Web Site: [www.sutherlandmedical.com.au](http://www.sutherlandmedical.com.au)