

"The only conforming Tubular Bandage Clinically Proven to heal Venous Leg Ulcers"
"Australian Multicentre open randomized control trial"**



sutherlandmedical

Tubular-Form™

Tubular Compression Bandage

Features:

- **100% Australian Made**
- Latex Free
- Product treated with Unique Low Fray Formula
- Easy to use Measuring Guides available
- Easy to use Applicators available



Measuring Guide for Application

Code	Size	Application	Roll	Circumference Measurement**
SM099	AA	Infant Limbs	10m	< 6cm*
SM100/SM200 ^	A	Infant Limbs	10m	7 – 11cm*
SM101/SM201 ^	B	Small Limbs	10m	12 – 14cm*
SM102/SM202 ^	C	Adult Limbs	10m	15 – 25cm*
SM103/SM203 ^	D	Large Limbs	10m	26 – 35cm*
SM104/SM204 ^	E	Adult Legs	10m	36 – 44cm*
SM105/SM205 ^	F	Large Legs	10m	45 – 50cm*
SM106/SM206 ^	G	Large Thighs	10m	51 – 60cm*
SM107	H	Extra Large Thighs	10m	61 – 70cm*
SM108	J	Small Trunk	10m	71 – 75cm*
SM109	K	Medium Trunk	10m	76 – 80cm*
SM112	K2	Medium Large Trunk	10m	81 – 84cm*
SM110	L	Large Trunk	10m	85 – 90cm*
SM111	M	Extra Large Trunk	10m	> 90cm*



*When applying to limbs, measure to fit the narrowest part. When applying to other areas of the body measure the affected area

^ 100 Series indicates Natural Colour 200 Series indicates Flesh Colour **Guide is for Medium Compression; for Low Compression select next size bandage up.

Fibre Content: Cotton, Elastodiene polyurethane and Polyester

ARTG No.132077

*Randomized clinical trial of three-layer tubular bandaging system for venous leg ulcers.
 Weller CD, Evans SM, Staples MP, Aldons P, McNeil JJ.
 Wound Repair Regeneration. November-December 2012, Volume 20, Issue 6

Sutherland Medical Pty Limited
 8 Kingston Town Close, Oakleigh, Victoria
 Ph: 1300 664 027 Fax: 1300 664 028
 Email: orders@sutherlandmedical.com.au
 Web Site: www.sutherlandmedical.com.au

How to apply 3 layer graduated compression using Tubular Form™ tubular compression bandage...

Why would I use 3 layers of Tubular Form™ ?

The compression achieved using 3 layers of straight Tubular Form™ may improve healing rates in people with venous ulcers, due to improved patient compliance by using a comfortable and affordable system that is easy for nurses to apply.¹

Tubular bandages are useful for managing patients who cannot tolerate other therapeutic forms of compression.²

How do I measure for the correct size Tubular Form™ ?

- Always measure the smallest circumference (ankle) to select the correct Tubular Form™ bandage size, and refer to sizing table below.

15 - 25cm
Size C
SM102

26 - 35cm
Size D
SM103

36 - 44cm
Size E
SM104

45 - 50cm
Size F
SM105

51 - 60cm
Size G
SM106

*Tubular Form Natural codes stated (also available in flesh colour)

**Sizing for moderate compression shown – to achieve low compression choose the next size up

How do I apply three layers of Tubular Form™ ?

Cut one long length, one medium length and one short length all from the same size Tubular Form™ and apply as indicated below.



MEASURE FROM BASE OF TOES TO JUST UNDER THE KNEE
STEP 1 – APPLY LONG LENGTH (1st LAYER)



MEASURE FROM BASE OF TOES TO MID CALF
STEP 2 - APPLY MEDIUM LENGTH (2nd LAYER) OVER THE TOP OF 1st LAYER



MEASURE FROM BASE OF TOES TO MID POINT BETWEEN ANKLE AND MID CALF
STEP 3 - APPLY SHORT LENGTH (3rd LAYER) OVER THE TOP OF PREVIOUS TWO LAYERS



COMPLETED THREE LAYER APPLICATION
*Colours are only shown to identify the 3 layers

What if the patient feels uncomfortable and cannot tolerate three layers of Tubular Form™ ?

- Simply remove the **SHORT** layer first. If a patient is still uncomfortable remove the **MEDIUM** layer however reapply each layer when the patient is feeling more comfortable.

What if I am unsure of the patients vascular condition?

- Always consult a physician if unsure of the patient's vascular condition.
- This method of compression therapy is recommended for patients with Venous Insufficiency.

1. Weller CD, Jolley D, McNeil J. Sub-bandage difference of Tubular Form and Short Stretch compression bandages: in vivo randomised controlled trial. Wound Practice and Research. 2010;18(2):100-4.
2. Bale S, Harding KG. Managing patients unable to tolerate therapeutic compression. Br J Nurs. 2003 Oct;12(19 Suppl):S4-13.

Sutherland Medical Pty Limited

8 Kingston Town Close, Oakleigh, Victoria

Ph: 1300 664 027 Fax: 1300 664 028

Email: orders@sutherlandmedical.com.au

Web Site: www.sutherlandmedical.com.au



sutherlandmedical