

Proudly distributed by:



sutherlandmedical

“This is the first device I have found that can actually take my stockings off, as well as put them on”

Judy McAllister, retired nurse,
Brisbane, Australia.

What is StockEZEon?

StockEZEon is a simple device that enables patients to **apply and remove** compression stockings independently. The unique design eliminates pain and potential skin damage as the hosiery is not drawn across skin and wounds.

Use of Compression Stockings – a continuing problem

- Many users physically struggle to apply and remove compression stockings
- In many cases, pain and further skin damage results.
- Compression therapy non-concordance is common, leading to longer treatment episodes and repeated recurrence of the condition
- Twice daily visits to apply and subsequently remove compression stockings add significantly to community nurse health costs.
- Available carer resources fall far short of patient demand for assistance.

StockEZEon is a comprehensive design solution:

- an easy way to put stockings ON and take stockings OFF independently
- adjustable to the patient's leg
- extendable handles
- does not require patients to reach their feet
- enables mobility – take it with you when you travel

Product Code for ordering:

SM264

Calf Measurement 39 -44cm

StockEZEon



StockEZEon Benefits

- increases user confidence
- reduces pain and skin damage
- eliminates or reduces the need for carers to be present to apply and remove stockings,
- significant savings in acute care and community nursing costs
- enables patient independence and improves quality of life
- improves compression therapy concordance rates, resulting in shorter treatment episodes, lower recurrence rates, and lower community health costs




Rob Skerman is the Managing Director of StockEZEon Pty Ltd. He has a consulting background in hospital systems, nurse staffing, patient workloads and aged care facilities. The StockEZEon invention was driven by the need for a more effective method of applying and removing compression hosiery. Key goals included enabling independence from carers, reducing pain and skin damage, and improving quality of life for mature age people. Rob has tertiary qualifications in Science (Microbiology) and Business Administration.

Application



Set the size of the device to your calf measurement

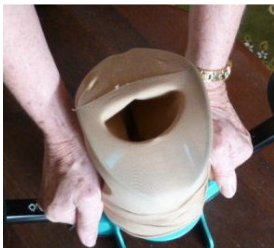

Slide the stocking over the top rim until the heel appears



Use the side clamps to hold the stocking while inserting your foot



Release clamps, pull upwards to deposit the stocking on to your leg

StockEZEon features

The unique StockEZEon system:

- Enables self management of APPLICATION and REMOVAL of compression stockings
- Minimises or eliminates pain from compression stocking use
- Size adjustable to the leg, for easier loading of stocking
- A unique top rim that also assists loading
- Side clamps on extendable handles – no need to reach feet
- Patients can stand in the device, aiding stability and safety
- Handles are removable for carer use and arm sleeve applications
- Easy to clean design
- Easily dismantled, and transportable in a carry bag

Removal



Turn down the stocking onto the top rim



Extend the handles, apply the side clamps, and push away from your leg

“I am no longer a burden on my family, my neighbours, nor do I need nursing assistance. It changes your whole lifestyle; you can get out and about and know you are not relying on anybody. I can take it with me; it just folds up quite neatly.”

Judy McAllister, Retired Nurse,
Brisbane, Australia.

To order contact

SUTHERLAND MEDICAL PTY LIMITED
Ph: 1300 664 027. Fax: 1300 664 028
Email: orders@sutherlandmedical.com.au

International Patents Pending