



sutherlandmedical

Aluminium Finger Splint



Application Instructions

1. Prior to application of Aluminium Splints, ensure that any open wounds are clean and dressed
2. Select an appropriate size Aluminium Finger Splint.
3. Bend the Aluminium Finger Splint to the desired shape to suit the injury.
4. Secure the Aluminium Finger Splint into position using either an elastic adhesive border or cohesive bandage.

Size Chart

CODE	APPLICATION	WIDTH	LENGTH	PACKET
FS12480-15	Finger Immobilisation	12mm	480mm	15
FS19480-15	Finger Immobilisation	19mm	480mm	15
FS25480-15	Finger Immobilisation	25mm	480mm	15
FS32480-15	Finger Immobilisation	32mm	480mm	15

***100% Australian Made**

***1.2mm Aluminum strip thickness**

***No sharp ends or edges, finger splints are rounded and bevelled**

***Padded with a soft breathable foam**

Please Note: Aluminium Fingers Splints are non autoclavable and are designed as a single patient use item only.

Sutherland Medical

8 Kingston Town Close Oakleigh Victoria 3166
P.O. Box 1194 Huntingdale Victoria 3166

Phone: 1300 664 027 Fax: 1300 664 028 Email: orders@sutherlandmedical.com.au
Or visit our website on: www.sutherlandmedical.com.au