



sutherlandmedical

Introducing **Trio Silken**<sup>®</sup>:  
silicone stoma gel for one piece  
removal leaving no residue



Feel the difference, experience the benefit of  
our advanced silicone technology.....

OSTOMY CARE



sutherlandmedical

# Silken<sup>®</sup> silicone stoma gel

## Advanced Silicone Technology

Have you ever wished that you could remove your hydrocolloid paste or strip in one piece leaving no residue? We thought you should be able to, so **Trio** developed **Silken<sup>®</sup>**.

**Trio Silken<sup>®</sup>** is a silicone gel that is used to fill skin folds, creases and uneven areas around the peristomal skin to provide a better surface for the application of an ostomy appliance. Unlike hydrocolloid pastes or strips, **Trio Silken<sup>®</sup>** will not absorb moisture or body waste and will not break down, leaving you with a difficult and messy substance to remove. Instead, as wear time progresses, **Trio Silken<sup>®</sup>** cures to a single piece of silicone adhesive that removes in one piece when fully cured.

Being sting-free on application and allowing your skin to breathe, **Trio Silken<sup>®</sup>** remains soft and flexible to ensure comfort. **Trio Silken<sup>®</sup>** can be used to create a perfect custom seal around the stoma, particularly useful for irregular-shaped or difficult to manage stomas.

Benefit from the improved quality of life through the avoidance of effluent leakage and consequential skin excoriation and faecal malodour - try **Trio Silken<sup>®</sup>** soft silicone stoma gel and be amazed at how easy it is to remove!

### Features:

- Fills skin fold creases and uneven areas around the stoma
- Creates a smooth surface for easy pouch adhesion
- Will not sting on application
- More hygienic than traditional hydrocolloid products- does not absorb faeces or urine
- Remains soft and flexible to ensure comfort
- Allows the skin to breathe
- One piece removal – leaves no residue
- Atraumatic removal – less skin irritation

Code	Description	Qty	SAS Code
TR1070	Silken <sup>®</sup> Silicone Stoma Gel	60g tube	80056L

### Sutherland Medical Pty Limited

8 Kingston Town Close, Oakleigh, Victoria  
Ph: 1300 664 027 Fax: 1300 664 028  
Email: [orders@sutherlandmedical.com.au](mailto:orders@sutherlandmedical.com.au)  
Web Site: [www.sutherlandmedical.com.au](http://www.sutherlandmedical.com.au)