



sutherlandmedical

Introducing **Tubular - Plus™**:  
A deluxe alternative for skin  
protection.



PROTECTION BANDAGES

# Tubular - Plus™

## Tubular Padded Protection Bandage

**Tubular - Plus™** is ideally designed to provide a protective layer in direct contact with the skin.

**Tubular - Plus™** is used for:

- Protection of the skin prior to the application of fibre-glass and plaster of paris casts.
- Padding to Limbs under compression bandaging

### Features:

- 100% Australian Made
- Cotton 98% (Elastodiene / Nylon 2%)


### Application:

It is important when measuring **Tubular - Plus™** prior to application, that an amount of 2.5cm is allowed at either end of the bandage.

The 2.5cm of additional bandage will gently roll after being cut to prevent airborne particles from entering the wound.

After using **Tubular - Plus™** for an extended period, the bandage will be subtly shaped to the limb it has been applied. Gently pull the bandage lengthways to remove the memory and the bandage will return to its original specifications.

Code	Size	Application	Roll	Limb Circumference
SM6629	5cm*	Small Limb	5m	20 – 31cm
SM6630	7.5cm*	Medium Limb	5m	30 – 50cm
SM6631	10cm*	Large Limb	5m	48 – 70cm
SM6632	12.5cm*	Extra Large Limb	5m	65 – 80cm
SM6633	15cm*	Large Thigh	5m	75 – 105cm



\*Size = unstretched width

Fibre content: Cotton Elastodiene, Nylon

### Sutherland Medical Pty Limited

Unit 2, 195 Chesterville Road Moorabbin Vic 3189

Ph: 1300 664 027

Email: [orders@sutherlandmedical.com.au](mailto:orders@sutherlandmedical.com.au)

Web Site: [www.sutherlandmedical.com.au](http://www.sutherlandmedical.com.au)

ARTG 133289

Version 3 / Created 18/5/2020