




ALGINATE DRESSING


Excellent natural dressing for use on moderately to heavily exuding wounds

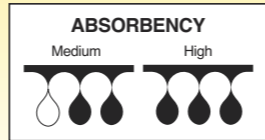

A B S O R B E N C Y



KIS Alginite Dressing is a sterile wound dressing which is composed of calcium alginate.

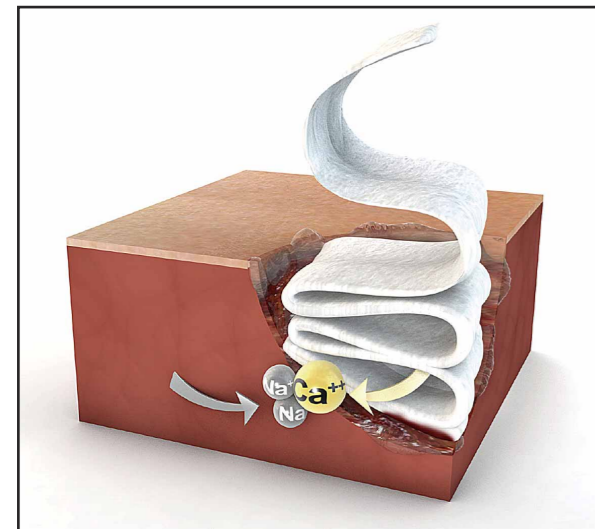

Mechanism of Action

When applied to a wound there will be an ion-exchange between the calcium in the dressing and sodium from the exudate, forming a soluble calcium/sodium alginate gel. The free calcium ions in the wound will amplify the clotting cascade and can reduce bleeding. The interaction with the wound exudates will create a protective gel that is nonadherent and creates a moist wound healing environment.

Highly Absorbent

It is capable of absorbing 10-20 times its weight and maintaining a moist wound healing environment.



Cavity Packing

The flexible and gentle soft fibre rope is ideal for the use of cavity wound packing.

Indications:

- Control minor bleeding
- Moderate to heavily exuding wounds
- Cavity wounds
- Pressure injuries
- Leg ulcers
- Post surgical wounds
- Graft and donor sites
- Skin tears

Tips for using:

- Clean the wound with sterile water or saline solution.
- Choose the right size of dressing according to wound area.
- Cut out dressing to size using sterile scissors.
- A secondary dressing is necessary when using calcium alginates.
- Before removing this product during a dressing change, dampen dressing with sterile water or saline solution to lower the risk of damaging the surrounding skin.

Benefits:

- Highly absorbent
- Maintains a moist environment
- Easy application and removal
- Excellent wet strength

Contraindications:

- Do not use this product when the wound is infected.
- KIS Alginite Dressing is not recommended to be applied to wounds that need surgery to achieve haemostasis.
- Do not use this product on dry wounds.
- Do not use on full thickness burns.

Size:

Variant	REF Code	Size
Sheet	SMAD050050	5 x 5cm
	SMAD100100	10 x 10cm
	SMAD125100	12.5 x 10cm
	SMAD200200	20 x 20cm
	SMAD050050E	5 x 5cm
	SMAD125100E	12.5 x 10cm
	SMAD050050S	5 x 5cm
	SMAD125100S	12.5 x 10cm
	SMAD025300S	2.5 x 30cm
Rope	SMAD002305R	2g

APPLICATION INSTRUCTIONS:

